axioprisa

reddot award 2021
winner

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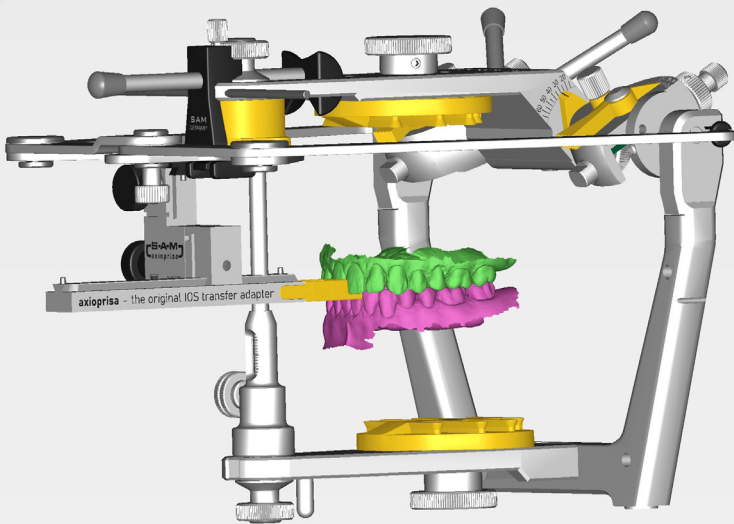
*Thank you Axioprism!
Now I know where
I am anatomical.*

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Problem: *The allocation of digital models right now is either mediocre, time-consuming or cumbersome.*

Goal: *A steric allocation in a fixed coordinate system - corresponding to the patient and matching the articulator.*

Solution: *the world's first scannable transfer fork assembly, specifically developed for use with an intra-oral scanner.*

Advantage: *The analogue proven method for anatomical transfer of the maxillary position into the digital articulator by using the bow.*

Conclusion: *No DVT. No lab scan. No expensive additional purchases. Logically comprehensible.*

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- Transfer bow -
e.g. SAM Axioquick

- Transferbogen -
z.B. SAM Axioquick

- Axioprisa -
scannable bite fork carrier

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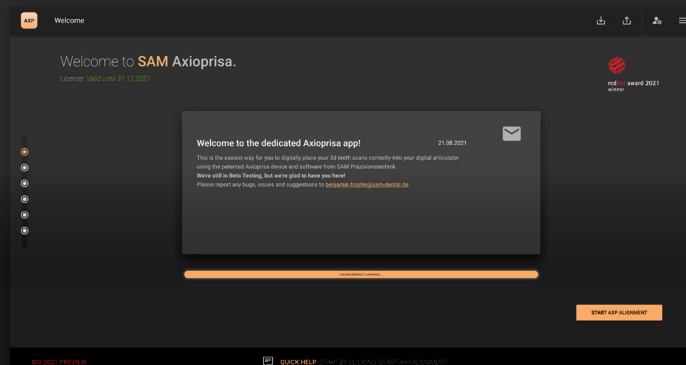
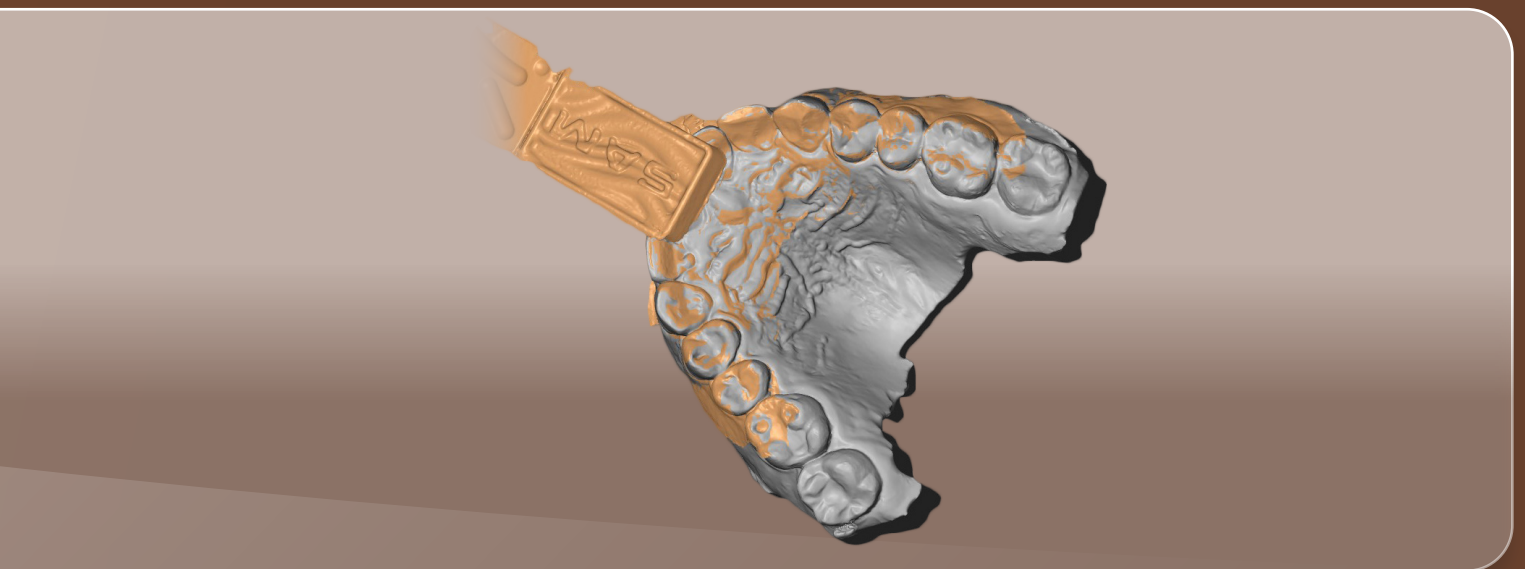
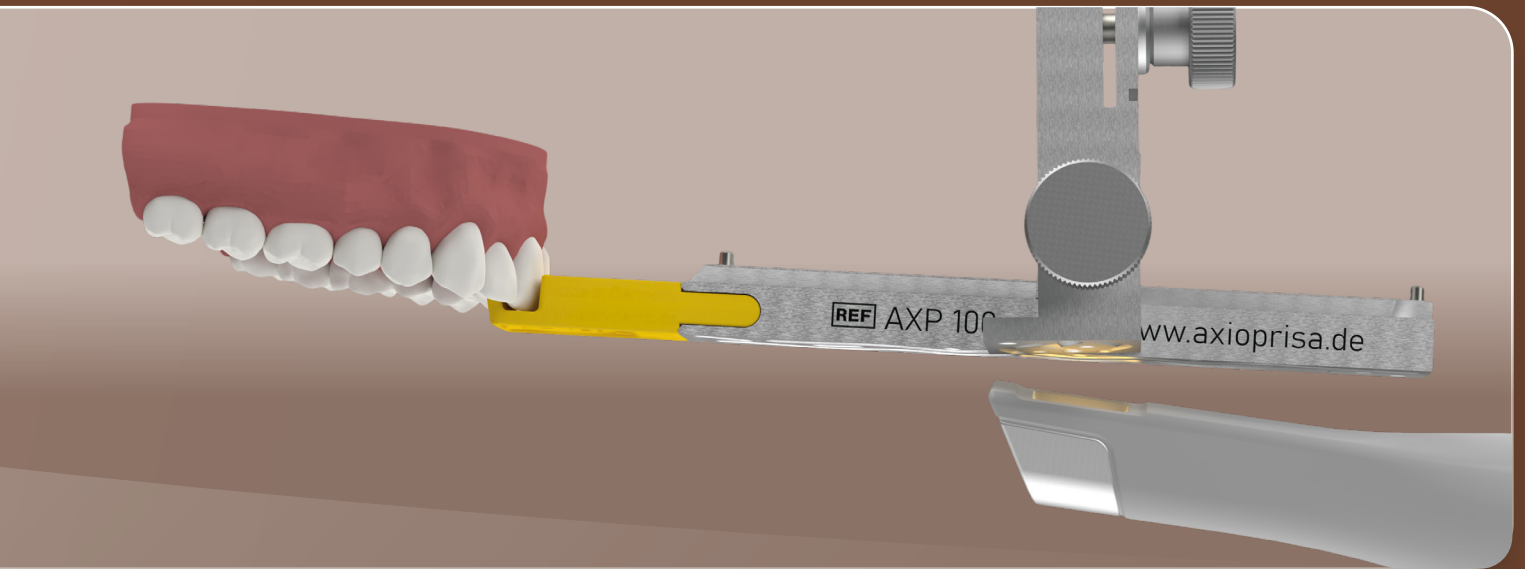
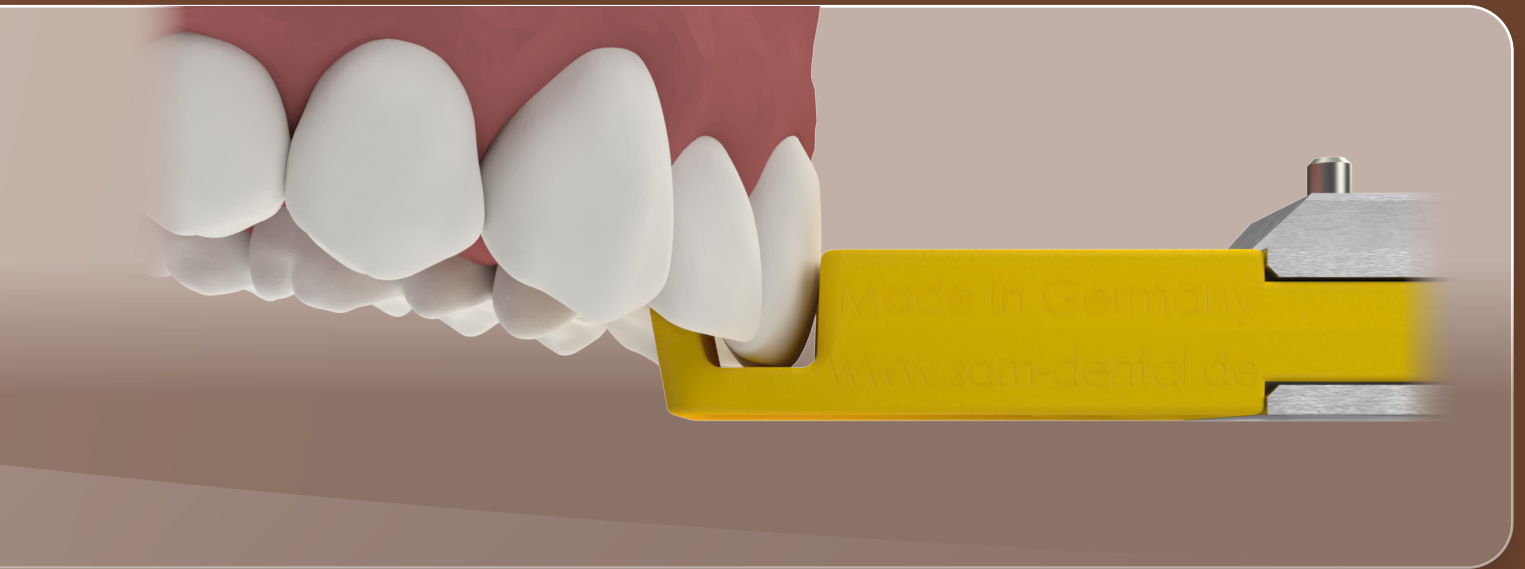
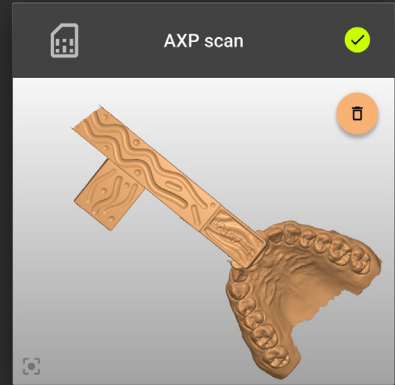
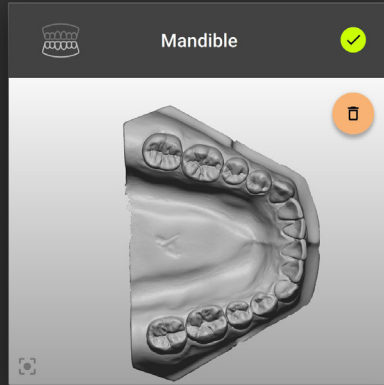
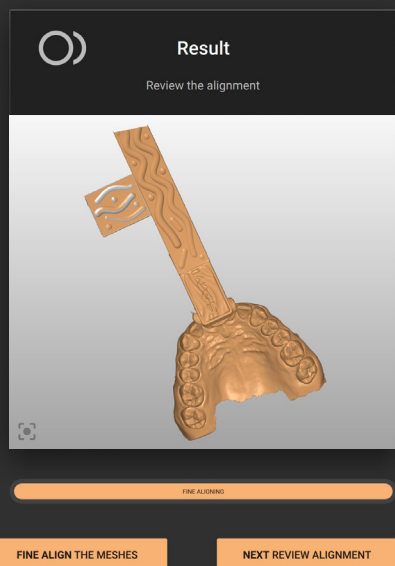
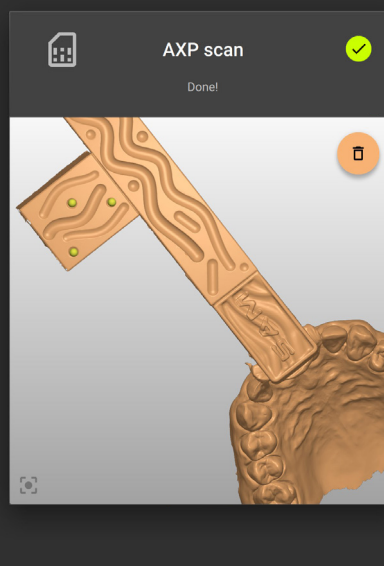
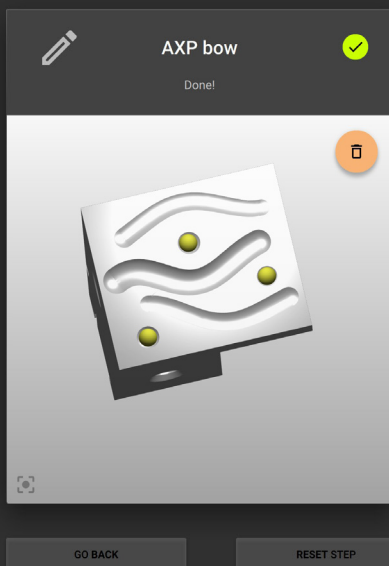
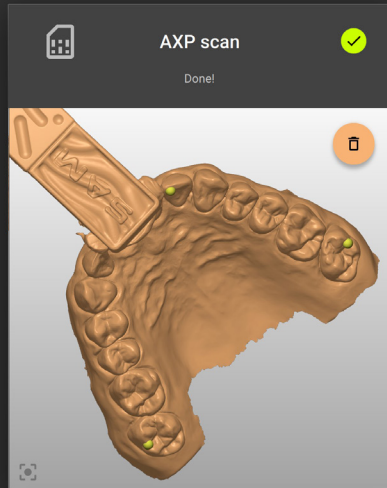
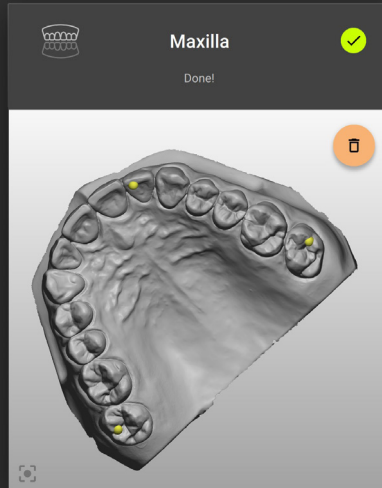
4

red dot award 2021
winner

Save as
articulator related STL file

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Digitization of the upper and lower teeth with an intra-oral scanner

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Digitization of the upper and lower jaw allocation, in maximum intercuspitation (HIKP) or centric relation (CR).

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Create an additional maxillary scan, e.g. as a pre-prep-scan with the 3shape Trios

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The electronic recording of mandibular movements and position in relation to a reference plane.

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Analog impression using hydrocolloid, alginate or silicone for the fabrication of traditionally established plaster models

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For positioning and stabilization of the tooth-to-tooth relationship in maximum intercuspitation (HIKP) or centric relation (CR).

OK-Position mittels Axiomatic *maxillary position with Axiomatic*

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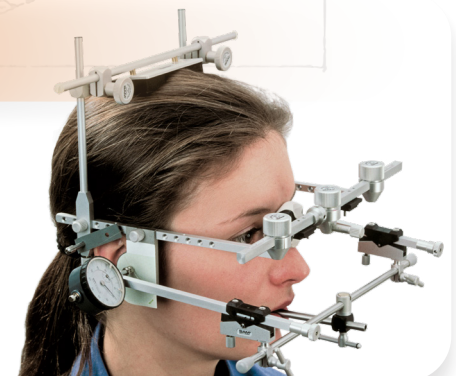
the traditional and safe transfer with the one-clamp transferfork assembly

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The original and the synonym for mechanical hinge axis determination and recording of mandibular movements since 1975

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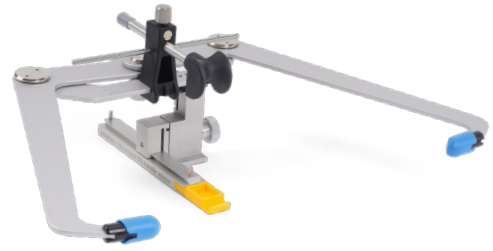
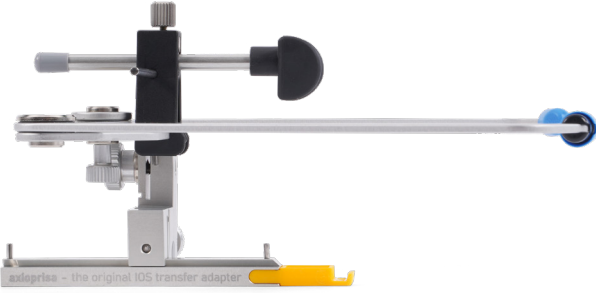
REF AXP 101L

500 St. / *Pc.*

TRANSFERBOGEN KIT TRANSFER BOW KIT

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Windows Kit
*with AXP software license for
24 months*

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Apple Kit
*with AXP software license for
24 months*

SOFTWARE-LIZENZEN SOFTWARE LICENSES

AXIOPRISA LIZENZ W12 AXIOPRISA LICENSE W12

12 Monate - Windows
12 months - Windows

AXP

AXIOPRISA LIZENZ A12 AXIOPRISA LICENSE A12

12 Monate - Apple
12 months - Apple

AXP

AXIOPRISA LIZENZ W24 AXIOPRISA LICENSE W24

24 Monate - Windows
24 months - Windows

AXP

AXIOPRISA LIZENZ A24 AXIOPRISA LICENSE A24

24 Monate - Apple
24 months - Apple

AXP

ONE STEP AHEAD

SAM Präzisionstechnik GmbH
Fussbergstrasse 1
82131 Gauting
Germany

www.axioprisa.de
patents pending