Dental Equipment and Products

***CATALOGO N.7***

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It is 1964 when Dottor Ticozzi founds the beginning company under his own name and starts developing, producing and selling products for dental labs. These are the booming years and the Ticozzi products, thanks their good quality, reach a top level position on the national market.

The years pass and the products and equipment line grows. To better manage the growing dimension of the company and its increased export business at the end of the 80's the company Ticozzi changes into Simed srl.

Today Simed sells in more than 30 countries all over the world covering more than 40% of its sales volume.

This catalogue's object is to better illustrate the complete range of the present production of equipment and products which covers an important part of the needs of the dental laboratory.

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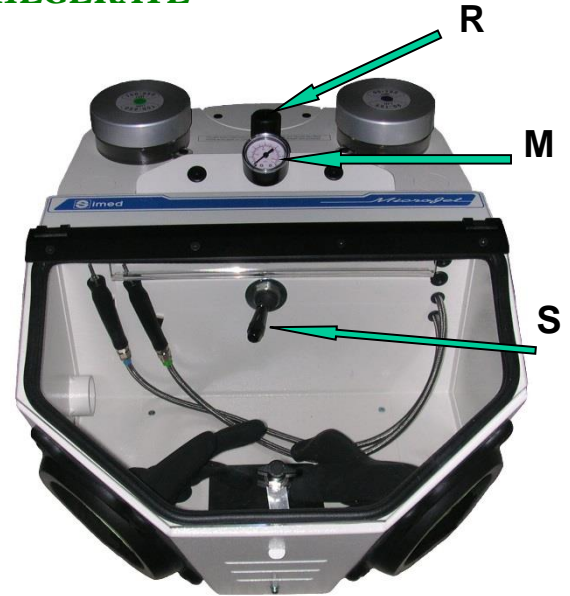
GB – Modular Sandblaster can be provided with 1 to 3 tanks customizable each one depending from the grain size used (50-80; 90-125 or 150-250 µm).

The choice of the tank (in case of more than 1 tank) can be easily done using a selector (S) positioned in the cabin. The sandblasting procedure is controlled by a pneumatic pedal.

The working pressure is controlled by the pressure regulator (R) and the value is indicated

by the pressure gauge (M). It is possible to require as optional one pressure regulator and one manometer for each container (RM). In this case it is possible to fix different pressure values for the single tanks, indicated by the respective manometer.

The connection to the exhauster is done through the hole placed on the side of the cabin. The blasting chamber has a led light and the plexiglas door is protected by a plastic sheet, which can be easily replaced.

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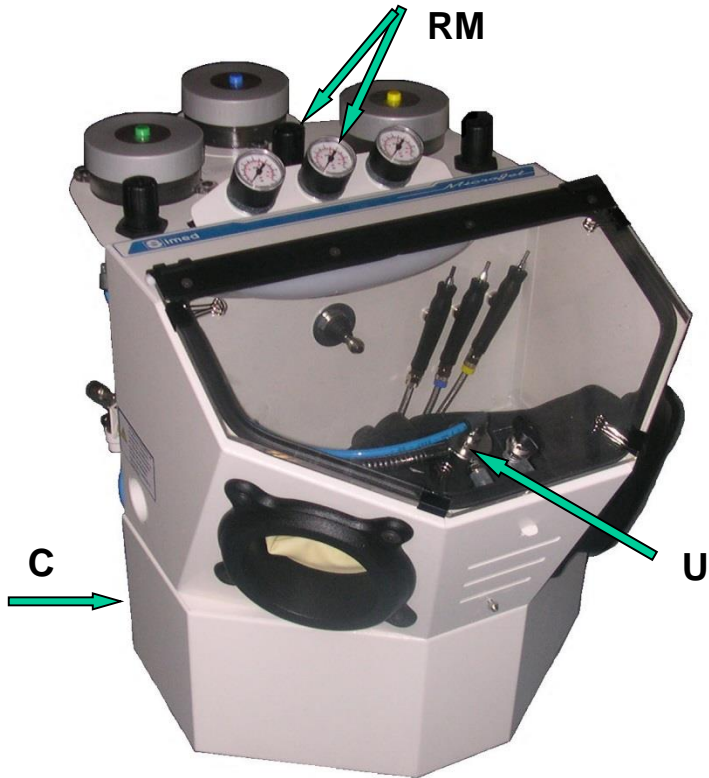
GB - The MICROJET can be also provided with a device for coarse sandblasting. In this case to the microsandraster is added a device (C) to collect the blasting media used. The collected sand is then reused through a 4mm diameter nozzle (U). This device allows to save abrasive and to have a stronger sandblasting power. In fact the unit works like two sandblasters and the choice between micro and coarse sandblasting is made by using two different pedals (one for coarse sandblasting, the other for microsandrasting).

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GB - Sandblaster provided with a recycling device for coarse sandblasting. It collects the abrasive falling on the bottom and reuse it. The sandblaster must be connected to an exhauster equiped with dust filters by inserting the exhauster pipe in the hole placed on the back of the cabin. The blasting chamber is well lit by a 9 Watt fluorescent lamp and the plexiglas door is protected by a plastic sheet, which can be easily replaced. The sandblasting procedure is controlled by a pneumatic pedal.

The suggested working pressure for coarse sandblasting is 6 bar. In the basic version the pressure is defined by the compressor while in the versions /P and /1 the operating pressure can be regulated by the pressure regulator and the value is indicated by the pressure gauge.

In the version /1 PRIMA can be provided with 1 tank for microsanding with aluminium oxide or glass beads in different grain sizes for the surface treatments of metal and ceramics. The selection between the tank for microsanding and the coarse sandblasting device is made by means of a practical selector placed outside the cab.

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(cod. **APRIMA**): PRIMA  
ca. 32 x 30 x 43 cm - ca. kg 9

versione base  
basic version  
version de base  
Basis Version

(cod. **APRIMP**): PRIMA/P  
ca. 32 x 30 x 43 cm - ca. kg 9  
con regolatore di pressione, manometro e filtro  
with pressure regulator, gauge and air filter  
avec filtre, manomètre et régulateur de pression  
mit Druckregler, Luftfilter und Druckanzeiger

(cod. **APRIM1**): PRIMA/1  
ca. 32 x 39 x 44 cm - ca. kg 10  
con l'aggiunta di 1 modulo per microsabbatura  
with an additional tank for fine sandblasting  
avec un réservoir additionnel pour microsablage  
mit zusätzlichem Tank zum Feinsandstrahlen

# SABBIATRICI AUTOMATICHE - AUTOMATIC SANDBLASTERS SABLEUSE AUTOMATIQUE - AUTOMATISCHE UMLAUFSTRAHLGERÄTE

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GB - Automatic and manual sandblasting unit.  
Equipped with rotating basket (capacity 8 - 10 frameworks) and sandblasting unit with flexible arm.  
Complete with air filter, pressure regulator, manometer, timer and led light.  
The sandblaster must be connected to an exhauster equipped with dust filters by inserting the exhauster pipe in the hole placed on the back of the cabin.

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GB - Work box with LED lighting.  
Possibility of connecting the external aspirator (not included) directly to the box in order to manage its start/stop with the lighting of the box.

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GB - Little fine sandblaster without cab, with one container. Complete with air filter, pressure regulator and manometer. Foot control or, on demand with tap (without foot control). On demand also possible with shoulder strap for transportation.

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GB - Dust extractor with double filtering system (paper bag and polyester filter). Max power 900 W with regulator. Max noise level approx 71 dB(A). Suitable with sandblasters and dry modeltrimmers.

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GB - Dust extractor with water-filtering and additional filtering of particles > 7 µm with a special cartridge filter. Max power 1200 W with regulator. Max noise level approx 77 dB(A) (with fonometer at 1 mt distance). Suitable with sandblasters.

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GB – The measurement unit of the grain size in the international system is the µm or micron (thousandth of millimeter). Mesh (number of meshes per square centimeter) is used too. In the metal-ceramic technique aluminium oxide should be used always pure to avoid pollution of the surfaces and occlusion of the micronozzles of the sandblasters.

SIMED aluminium oxide, available in the grit sizes 50, 80, 90, 110, 125, 150, 180, 250 µm, is 99,75% pure and it doesn't contain free silica.

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GB – GOLD SHYNE glass beads are highly resistant to wear and impact. They are used for surface cleaning and polishing in all those cases where it is necessary to avoid any superficial material removal or alteration of the treated objects. They are therefore particularly suitable for the treatment of precious alloys, resins and acrylic materials.

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GB – Corundum brown suitable for coarse sandblasting of metal surfaces. Available in the grit sizes 46, 60 and 80 mesh (350, 250, 180 µm).

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GB – 6 liter automatic duplicating unit equipped with an electronic board for a complete automatic control during the whole working cycle. The new software ensures lower electric consumption and more flexibility. The turning speed is reduced for a more gently melting process of the gel and the new three-phase motor grant more power and lower noise. Container, stirrer and lid made by stainless steel. It permits the delayed start of the cycle with weekly programming and the control of the age of the duplicating material. The special control of the stirrer ensures a uniform temperature inside the tank. In case of power failure the equipment starts again the cycle from the point it was interrupted except in case in the meantime the duplicating material solidify. In this case the equipment stops the cycle to avoid the motor to be overloaded.

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GB – High quality duplicating gel for all investments. High precision, elasticity and long life. Colour white. Shore A hardness: 15°.

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GB – High quality duplicating gel for all Investments. Great precision, toughness and life.

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GB – Duplicating gel for investment models with alcoholic binder. Precise, stable and elastic. Available white, pink and green.

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GB – High transparency duplicating gel for audiology applications, suitable for all light curing materials.

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Aluminium flask with rubber plate for gel duplication

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GB – Addition curing silicone for model duplication with excellent reproduction of details and minimal linear shrinkage (< 0,1 %). Working time approx. 5 min. Demoulding time approx. 30 min. Available in three different hardnesses and in a transparent version.

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GB – Special condensation silicone for laboratory use, with hardness 85° ShoreA or extra hard 95° ShoreA. Working time approx. 1 min 15 sec. Setting time approx. 3 min.

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GB – Addition curing silicone for laboratory use, with hardness 70°, 80° or 90° ShoreA. Higher precision and without any harmful component. Working time approx. 1 min 30 sec. Setting time approx. 3 min.

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GB - The vibrators of the VIBRO serie, developed for vibrating and compacting investments, plasters or similar materials, are equipped with a rubber plate, easy to remove to facilitate the cleaning operations, and an electronic power regulator that permit to modify the vibration intensity continuously.

The VIBRO 6 model is also equipped with a two-way switch for selecting a lower and a higher vibration.

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GB - Electrolytic CoCr alloys polishing unit with approx. 2,5 litres tank. The power generator creates direct current which allows by means of the tank solution (CROCOLIT), the electrolytic polishing of the element surface.

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GB - Electropolishing solution for CrCo alloys.

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GB – Burnout furnaces available in three versions: SF170, SF240 and SF285. Every model can be equipped either with the “/E” or the “/P” electronic board. All equipment can reach a final temperature of up to 1100 °C, have the possibility to preset the cooling phase, are prepared for the connection to an extractor equipped with intake fan controlled by the electronic board and have a safety microswitch that cuts off the power to the resistance in any phase of the heating cycle when the door opens.

The electronic board of the “/E” model controls a three-stage cycle with temperatures, rising speeds and standby times that may be preset by the operator and is equipped with a timer for deferred ignition which may be set up to 99 hours and 59 minutes.

The electronic board of the “/P” model can store up to 10 cycles, each of them including 4 stages with temperatures, rising speeds and standby times free presettable. It has also the possibility to preset an 8 stage cycle and is equipped with a weekly timer for deferred ignition.

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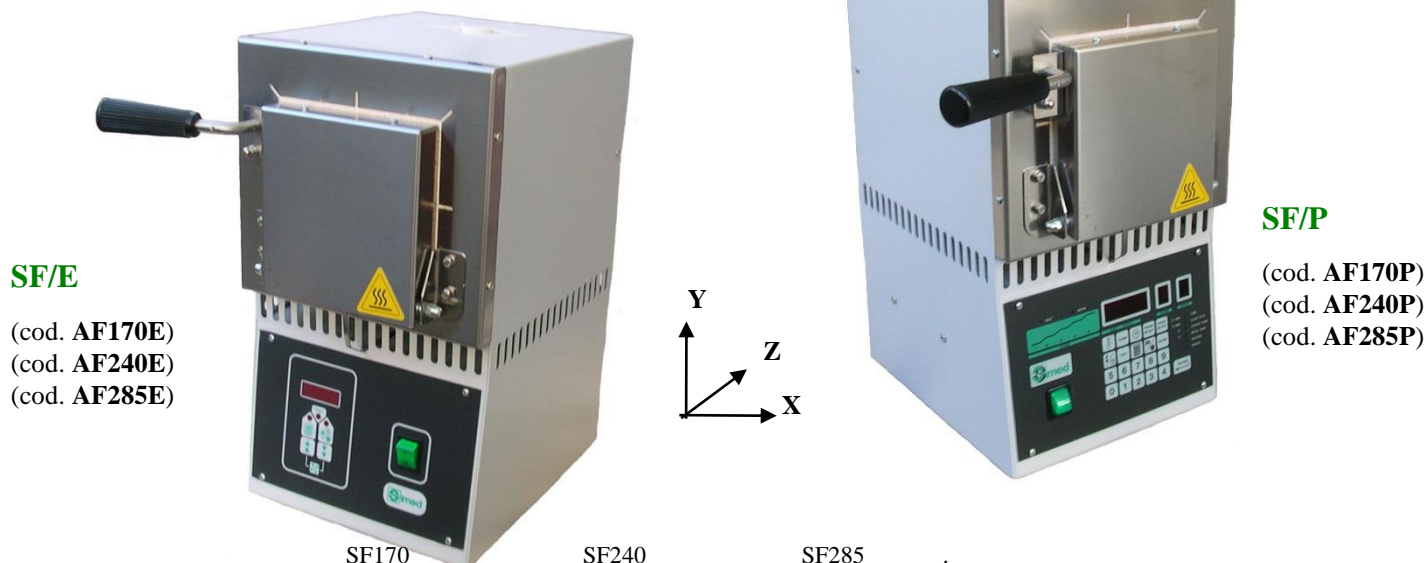
GB – Phosphate bonded investment for cobalt chrome frameworks.

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GB – Phosphate bonded investment for cobalt - chrome frameworks.

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GB – Investment with chemical binder for cobalt chrome frameworks.

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GB – High precision phosphate bonded investment for bridge and crowns. Suitable for precious and non-precious alloys as well as for the pressable ceramic technique for metal free crowns, inlays, onlays and veneers.

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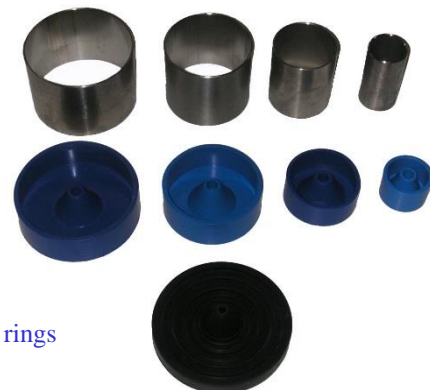
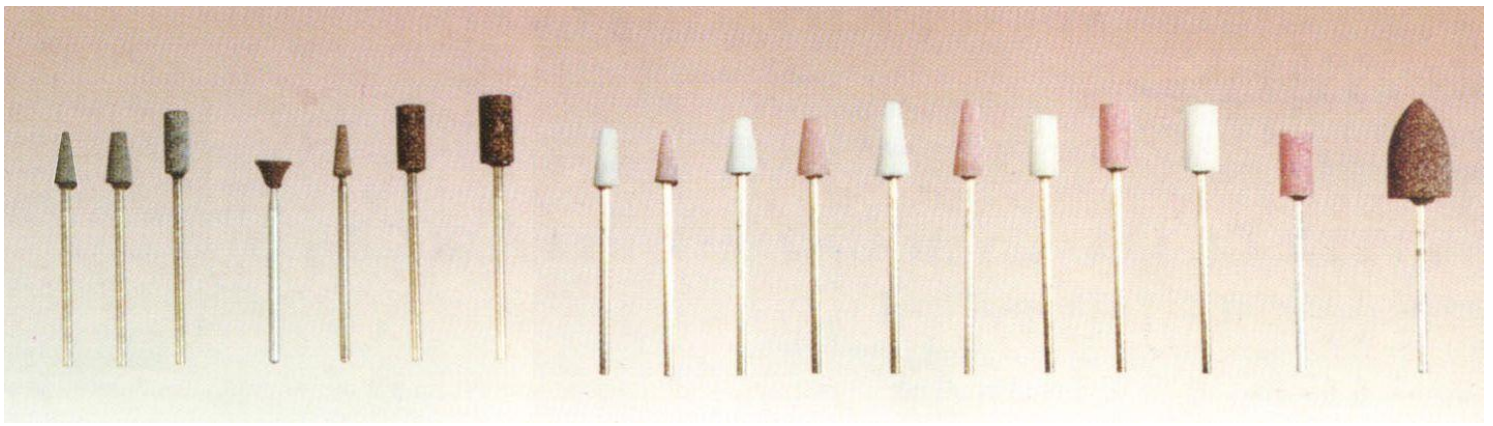
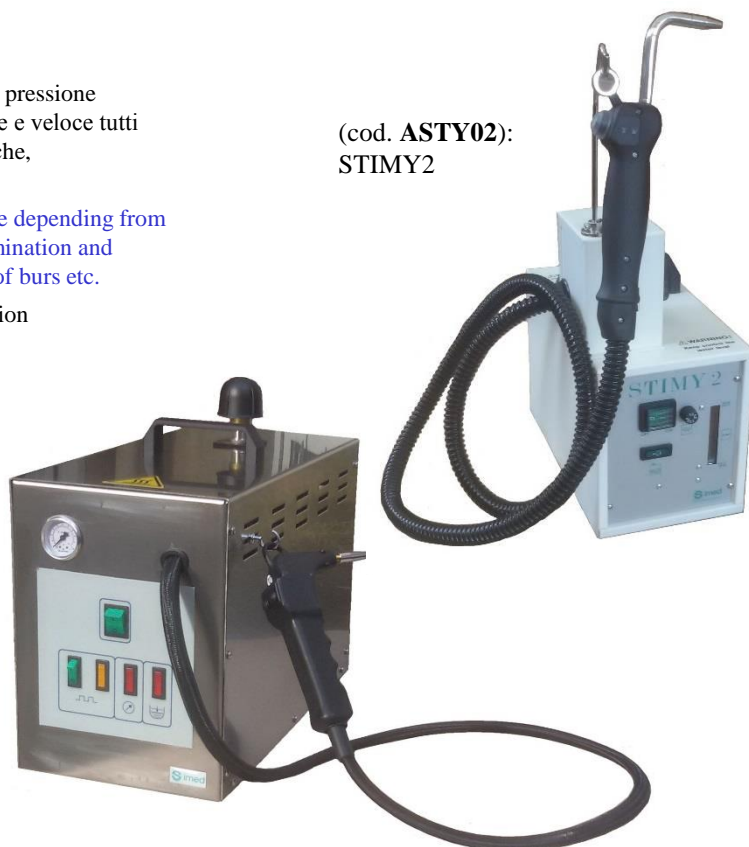
GB - Steam generators with different tank capacity and operating pressure depending from the model. Fulfil safely and efficiently all cleaning needs, from decontamination and degreasing of metal castings, to removal of investment residue, washing of burs etc.

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GB - Model trimmers 2800 rpm, 1000 W.  
Available wet, disk cooling with water, or dry with diamond disc.  
The wet version H2O is equipped with solenoid valve and tap-flow regulator.

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GB – Light curing unit suitable for all light curing composites.

The alternate position of the 4 + 4 9W fluorescent lamps, together with the 2 x 75W dichroic lamps, grant a uniform light emission of UV-radiation and white light covering the light emission range needed by the light curing materials (320-550 nm).

LIGHT MASTER is equipped with timer, turning support to optimize the exposition, fan to reduce the internal temperature and hourcount to check the lamps life.

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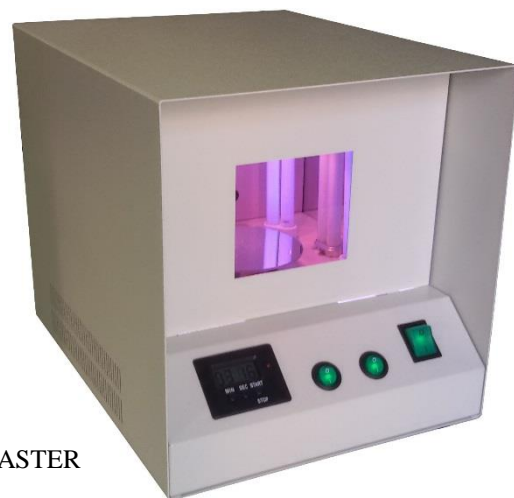
Schermo utile V30: 30 x 16,5 cm; V38: 38 x 20,5 cm

GB – Negatoscopes placeable on the wall or on the table in horizontal, vertical or inclined position, using the support supplied with it.

The lamps are in accordance with clinical prescriptions that requires a very white light (6500° Kelvin) for a precise reading of details. The screens are in plexiglas.

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GB – Stereoscopic trinocular microscope for gradual magnifications from 7x to 45x. Working distance approx. 100 mm.

The microscope mounts 2 eyepieces WF 10x , zoom objective 0,7 - 4,5x and a mounting port for photo attachment and CCD video attachment.

Optionals:

- Anular illuminator to be mounted on the objective
- Eyepieces 20x
- Objective 0,5x
- Protective lens

### KIT UNIVERSALE INTERFACCIA SMARTPHONE- MICROSCOPIO DEVICE FOR SMARTPHONE-MICROSCOPE CONNECTION

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GB –Thanks this device it is now possible to easily and economically connect every smartphone to take pictures or movies through the microscope.

To connect the smartphone the system offers a universal support or specific cases according to the smartphone used.

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**NOTE: THE DEVICE IS COMPATIBLE WITH EVERY  
MICROSCOPE with ocular pipes diameter 30 or 23 mm!**

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GB – This head loupe comes with 5 changable lenses. This way, the magnification is variable between 1x and 6x. The loupe has a built-in, removable LED illumination which is powered by three AAA-batteries (included).

# PULITRICI - POLISHING MACHINES - TOURS A POLIR - POLIERMOTOREN

## P 3/2 & P 5/2

(cod. **APUL32**): Pulitrice P3/2 300W  
1400/2800 min<sup>-1</sup> ca. kg 6,5

(cod. **APUL52**): Pulitrice P5/2 500W  
1400/2800 min<sup>-1</sup> ca. kg 10,5

## PASTA IN PANI - POLISHING COMPOUNDS - PATE A POLIR - POLIERPASTE

(cod. **APB001**): Bianca per lucidare leghe al CrCo  
White for polishing CrCo-alloys

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(cod. **APM001**): Marrone per lucidare oro e resina  
Brown for polishing gold and acrylics

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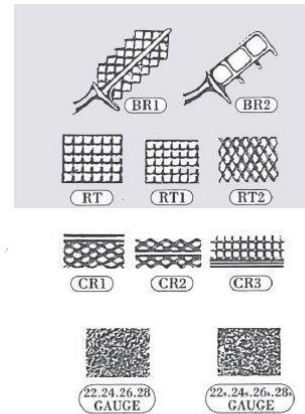
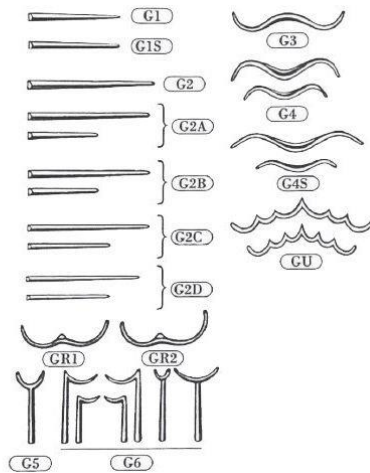
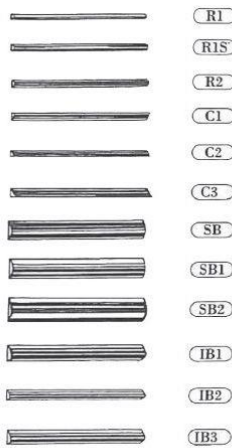
GB – Solvent for dissolving plaster, alginate etc. from aluminium, stainless steel or plastic trays.

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GB – Powder, neutral pH. Dissolved at 10% in water eliminates alginate from aluminium impression trays.

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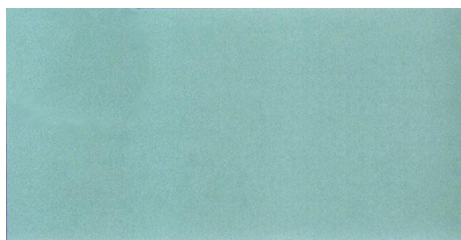
Diametri disponibili: 2,0 - 2,5 - 3,0 - 3,5 - 4,0 - 5,0 mm

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## TAZZINE IN SILICONE - SILICONE CUPS BOL EN SILICON – SILIKON ANRÜHRBECHER

(cod. **VTS001**): Piccola - *Small* – *Petit* – *Klein* 15 ml

(cod. **VTS002**): Media - *Medium* – *Moyen* – *Mittel* 25 ml

(cod. **VTS003**): Grande - *Large* – *Grande* – *Gross* 65 ml

## **Simed srl**

Via Monte Spluga, 28

20021 Baranzate (MI) - Italy

Tel.: +39 023561518 - +39 023562314

[info@simedonline.com](mailto:info@simedonline.com)

[www.simedonline.com](http://www.simedonline.com)