




With over 100 years of commitment to quality, we prioritise safety and reliability. To meet the diverse needs of our customers, we offer a comprehensive range of products.

# Dental Products

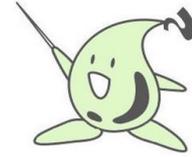


## Dental impression disinfectant & stabilizer

### Chlor Anchor

Proper hygiene measures must be taken because invisible germs still pose a threat. Chlor Anchor supports standard precautions and helps maintain hygiene in dental clinics. The impression model is the blueprint for prosthesis fabrication. Chlor Anchor stabilizes the dimensional changes of impression materials.



Tablets: 12g x 30 packs/box

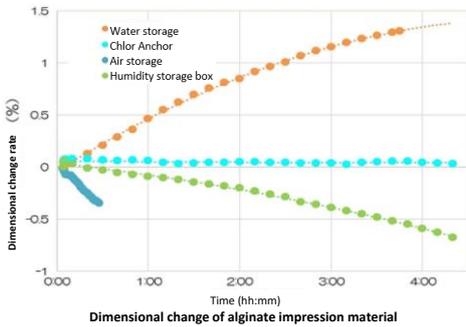

Disinfection and dimensional accuracy stabilization of alginate impression materials

#### Features

- Easy to Use**  
 Tablet form that dissolves when placed in water, releasing chlorine. Can be added to water without direct contact.
- High Chlorine Concentration for Disinfection**  
 Dissolving one tablet in one liter of water results in a chlorine concentration of 1300 ppm.
- Stabilizes Impression Dimensions**  
 Immersion for approximately 3 hours keeps dimensions stable.

#### Usage Instructions

- Put 1 liter of room temperature water in a plastic container and add one Chlor Anchor tablet. Stirring will speed up dissolution.
- Rinse the impression with running water and remove excess water.
- Immerse the entire impression in the Chlor Anchor solution.
- After 15 minutes to 1 hour, remove the impression and rinse with water.
- Remove excess water and pour in the plaster. Processing with Smooth Maker before pouring will result in a smoother plaster model.

## Dental impression lubricant

### Smooth Maker

Smooth Maker improves the surface properties of gypsum models when used on impression bodies after taking impressions. It is effective not only on alginate impressions but also on combined agar-alginate impressions.




200 g spray bottle

Smoother for dental impressions

#### Features

- Improves Surface Properties**  
 Effective on both hard plaster and super hard plaster that is prone to surface roughness, reproducing smooth impression surfaces.
- Easy to Use**  
 Spray type, can be used by spraying.

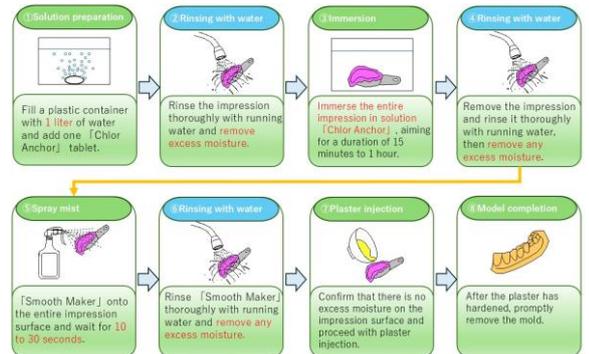

For inquiries and product orders, please contact the following



Manufacturer and Distributor  
**SAN ESU GYPSUM CO.,LTD** <https://www.sanesugypsum.com/dental/>